



Senior friendly products



KC(Korea Certification)



Q-Mark Certification



Venture company



Certification

ISO 9001, 14001, 13485 Certification

High-tech walking aid for better life

Nice Walker

Owner's Operator & Maintenance Manual

MODEL
4SE





Model Name

Nice walker 4SE

Please read it carefully to operate your Rollator safely and properly before use. Keep this manual as a future reference as well.

GENERAL WARNINGS & SAFETY INSTRUCTIONS

1. You MUST grab the handles properly when in use and try NOT to lean on the rollator when not in use.
2. Please MAKE sure that the brakes MUST be locked when not in use.
3. Please do NOT let children play with this rollator.
4. This product is designed for indoor and outdoor use. If possible, it is better for indoor. In particular, when using it for outdoor, it's always recommendable to check the ground condition for your safety, if the ground surface is too rough, or if any stumbling blocks on the road, etc.
5. Do not use rollator to climb stairs, or to walk backwards or to go over obstacles.
6. Check the brakes for proper operation. Make sure that all parts are secure and that moving parts are in good working order.
7. In case of using the rollator at night time or in any dark condition, please be advised that reflecting tapes, reflectors and safety lights are attached for your safety.
Be careful if any stripped parts of tape, damages or invisibility.
In particular, since the safety lights are very fragile and weak against any impact or moisture, be aware of not to get it wet and damage all the time.
8. Do not exceed the maximum weight capacity or serious injury could result.
*Rollator : 130kg(286lbs) / *Basket : 10kg(22lbs)
9. Please make sure all the parts are fixed in the right position before use.
10. Check if any parts and screws are damaged or loosened before use.
11. Do NOT wet the rollator in rain or keep away from the direct sun light and make sure keep it clean for the long period of time when not in use.
12. Since the rollator has some non-durable components, such as rubber accessories, brakes and grips and any parts related to Auto Brake System , it is always recommendable to replace after certain period of time of use in order to operate the rollator properly and normally.
13. All wheels must be in contact with the floor or ground at ALL Times. This will ensure the rollator is fully balanced.
14. Do NOT have anyone push you while you are seated on the Rollator. This is a walking aid only, not as a transportation device.
15. Try NOT to move forward with rollator when it is on parking mode. Be sure to unlock the parking mode by pulling the brake lever up or it might get your rollator seriously damaged.
16. Do NOT use any accessories or parts, not authorized by Nicetech, for use with this rollator specifically.
17. This product has the right and ownership of invention patent, industrial design registration and All types of required registration of ownership. Please note that it would be harm and possibility of any loss, in the event of any unauthorized duplication or replication of the product.

4SE
MODEL

Assembled Rollator



1. Remove carefully the rollator from the shipping carton.
2. Examine all parts for shipping damage. Please DO NOT use the Rollator if any apparent damage or any parts are missing.
3. Make sure all nuts, bolts and screws are tightly fastened.
4. Make sure all 4 wheels are in good working order and condition

* To improve performance, it is subject to change some specifications without any prior notice



4SE OPENING THE ROLLATOR

- After unhooking the fixing hook from the supporting pipe, Make sure that the rollator is to be fully extended to the side. Then, push the seat support pipe down slightly until you hear the 'click' sound.
- Make sure that all 4 wheels contact with the floor and the rollator is stable in the open position.



4SE FOLDING THE ROLLATOR

- Pull up the seat holder slightly and fold it.
- Fix the rollator by using fixing hook, as you see the picture.



4SE ADJUSTING THE POSITION OF THE HANDGRIPS

- After opening the rollator, please be advised that both handgrips are adjustable in height at the user's preference. To adjust the height of handgrips, remove the height adjustment nuts under the handgrips. Then, slide the handgrips up or down until they are at the proper height and screw the height adjustment nuts to the frame. Make sure that all nuts are fully tightened and not moving from side to side. The both heights should be equal.



Height Adjustment Nut



Small Round Button

4SE OPERATING THE PARKING BRAKES AND BRAKE HANDLES

- Pressing the round parking buttons on each side will lock the wheels in place.
- Pulling up the brake levers slightly will unlock the wheels from the parking position.
- Pulling slightly up the brake levers, locks the wheels momentarily from moving mode(or To slow down, squeeze the brake levers up)



Parking Brake



Unlocking the parking brake

4SE CHECKING AND ADJUSTING THE BRAKES

- To check if the brakes are working normally, be sure if you don't hear any squeaking sound when you let it roll on N(neutral) position without any brake systems on. If any squeaking sound, you can adjust the brakes by tightening up the brake control nuts, 1 or 2 times in a counterclockwise. (please see the figure)



Brake Control Nut



4SE LEFT AND RIGHT WHEELS OF SPEED CONTROL METHOD



Speed Control Handle

- This rollator has two different major functions of walking method. Turning Control Knob to 'D' position works with this unique electronic Auto Brake Device and Backward Speed Reduction Device, as the electronic lights are on with its own generator system.
- Be sure that it must be at same 'D' position on both wheels by turning Control Knob slightly until you hear the 'click' sound. This high tech device of speed resistance system prevents the acceleration of rollator on walking downward slope, and creates the reduction of speed and prevents from the phenomenon of being pushed back on any stop when walking on upward slope.
- Turning Control Knob to 'N'(Neutral) position while all electronic speed control systems are turned off, is more recommendable for the person who is rather free to walk.

4SE SPEED REGULATOR



- Just on 'D' position of Control Knob, Speed Regulator Device is now working. On 'D' position, with the selection of speed level from 1 to 6 on speed regulator panel, depending on user's physical condition, it could either go faster or slower with this speed resistance systems on. Level 1 is slowest and it goes faster as the numbers are going higher. Everytime this system is on, the safety lights should be turned on except for moving backward.

4SE SPECIFICATIONS



- Model : Nice Walker 4SE
- Rollator Dimension : W600×L715×H780(up to 905)mm
- Material : stainless steel, steel, plastics, aluminum
- Maximum Weight Capacity : less than 130kg(286lbs)
- Receptacle bag : W240(300)×L240×H250mm
- Load capacity : 10kg(22lbs)
- Weight : 9.5kg(21lbs)
- Wheel : 4 wheel type
- Hand brake : L, R lever
- Electronic auto brake device : regenerative braking
- Backward speed reduction : regenerative braking
- Parking brake : hinge pin
- Speed control : neutral, low(6 step)
- Folding : L&R folding type
- Safety lamp : LED
- Origin : Korea

MODEL 4SE

Lifetime Limited Warranty

Your Nicewalker Rollator brand is warranted to be free of defects in materials and workmanship for a lifetime of the original consumer.

The device was built to precise standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service. In the event of a defect covered by this warranty, we will, at our option, repair and replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. This warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and tear. Non-durable components are warranted for (6) months.

If you have a question about your Nicewalker device or this warranty, please contact an authorized Nicewalker dealer.



Nice Walker, it is all about safety and convenience!

ISO 9001, 14001, 13485 Certification
NICE TECH

Nice
Walker



ISO 9001, 14001, 13485 Certification

NICE TECH

- Head office 6202 Daegu Health College Venture Business Incubation Center
7 san, Taejeon-dong, Buk-gu, Daegu Korea.
tel. **82-53-311-6568** fax. **82-53-314-6569**
- Factory 301-1 Gak san dong, Dong-gu Daegu Korea.
tel. **82-53-961-6568** fax. **82-53-961-6569**
- Homepage **www.nicetech.net**

Nicetech USA, Inc.

- Address 21356 Nordhoff St. Suite #108
Chatsworth. CA 91311 USA
tel. **818-527-1820** fax. **818-626-8611**
- E-mail **nicetechroger@gmail.com**
- Toll Free **1-855-641-7818**